ImmunoDiagnostics Limited

High Sensitivity C-Reactive Protein (hs-CRP) Control

Cat. No.: 51121-C1 Type: Control

KIT COMPOSITION

Control 1-2 $2 \times 1 ml$

INTENDED USE

This control is intended to be used as a consistent test sample of known concentrations for monitoring the *ImmunoDiagnostics Limited* High Sensitivity C-Reactive Protein assay

(Cat. No. 51121)

CONTROL PREPARATION

The controls are ready to use and do not require reconstitution.

STORAGE AND HANDLING

All controls should be stored refrigerated (2-8°C). Return all controls to 2-8°C promptly

after use. Unopened controls can be used for one year from the date of manufacturing as

indicated on the expiration date on the package and bottle labels. Opened bottles of

control can be used for 1 month if stored at 2-8°C.

CONTROL VALUES

Control 1: 0.9~1.1 mg/L

Control 2: 2.7~3.3 mg/L